# COLDAMARIS *Junior*

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# COLDAMARIS *Junior*

## Nasal and Throat Spray

### What is COLDAMARIS Junior Nasal and Throat Spray?

COLDAMARIS Junior Nasal and Throat Spray is an isotonic sodium chloride solution with dexpanthenol and is used for moistening the nose and pharynx as well as to support the treatment of a cold and allergies.

### What does COLDAMARIS Junior Nasal and Throat Spray contain?

COLDAMARIS Junior Nasal and Throat Spray contains sodium chloride and 2% dexpanthenol in an isotonic solution **without preservatives**.

Preservatives are not necessary thanks to the special spray system with filter.

### How does COLDAMARIS Junior Nasal and Throat Spray work?

COLDAMARIS Junior keeps the nasal and pharyngeal mucosae moist. Due to the ingredients it has an abirritant and regenerative effect.

A dry and irritated nasal or pharyngeal mucosa may result from general air pollution or frequent stays in places with a low relative air humidity such as heated or air-conditioned rooms, cars, trains, or airplanes. Both factors – air pollution and low air humidity – cause drying of the nasal and pharyngeal mucosae.

The filtering function of the nose – that is, its ability to hold back particles in respiratory air (dust, pollen, allergens, and others) – can only be performed optimally by a moist nasal mucosa with a functioning and healthy ciliated epithelium.

COLDAMARIS Junior keeps the nasal mucosa moist and thus guarantees the cleansing and filtering function of the nose.

### Therapeutic Indications – When should you use COLDAMARIS Junior Nasal and Throat Spray?

- For moistening in the case of dry nasal and/or pharyngeal mucosa (caused e.g. by dry air, pollution or Sicca-syndrome) and in the case of encrustation of the nasal mucosa.
- For supporting treatment in the case of a cold, running nose and allergic cold.
- For supplementing treatment when using decongestant medication.

COLDAMARIS Junior Nasal and Throat Spray is suitable for use in children of 1 year and older and in adults.

Use in infants below 1 year of age should only be when recommended by a physician.

COLDAMARIS Junior can be used in pregnancy and lactation without hesitation.

### Dosage and Method of administration – How often and how should you use COLDAMARIS Junior Nasal and Throat Spray?

Several times a day, depending on the individual requirement, spray one or several jets into each nostril or into the pharynx.

The application may be repeated as often as desired and may be prolonged for a longer period of time (up to 30 days).

Remove the protective cap before use and pump a few times if necessary until a uniform spray mist is emitted. COLDAMARIS Junior is now ready for use.

Keep the bottle as upright as possible when spraying. The suction tube must protrude into the liquid. Insert the spray nozzle carefully and not too deeply to avoid injury.

If the functionality of the spray nozzle is impaired, press the nozzle several times in an upright position until the spray mist is emitted again.

After use, put the protective cap back on the nozzle.

### When should COLDAMARIS Junior Nasal and Throat Spray not be used?

If you or your child are/is allergic (hypersensitive) to any of the ingredients of the product.

### Possible side effects/interactions

There are no known side effects to date.

There have been no studies on interactions with other products

### Precautions for safe use

COLDAMARIS Junior is a medical device. If the condition does not improve or pain occurs consult a doctor.

In case of heavy crust formation or inflammatory processes, make sure to consult a physician.

For reasons of hygiene and to avoid transmission of pathogens, each spray bottle should be used **by one and the same person** only.

Keep out of reach of children!

### Pack size

Bottle with 20 ml solution, with nebuliser.

1 puff = 0.05 ml.

### Shelf-life

COLDAMARIS Junior can be used until the expiry date specified on the package.

### Storage instructions

 Do not store above 25 °C.

 Protect from light.

 **Manufacturer:** MoNo chem-pharm Produkte GmbH, Leystraße 129, A-1200 Vienna - Austria

**Distribution:** SIGMAPHARM Arzneimittel GmbH, A-1200 Vienna - Austria

Revision date: September 2020